Faller


OSKAR

BEDIENUNGSANLEITUNG // OPERATING MANUAL
MANUEL D'UTILISATION // BEDIENUNGSHANDLEIDING
ISTRUZIONI PER L'USO // MANUAL DE INSTRUCCIONES


Quick Start Guide


1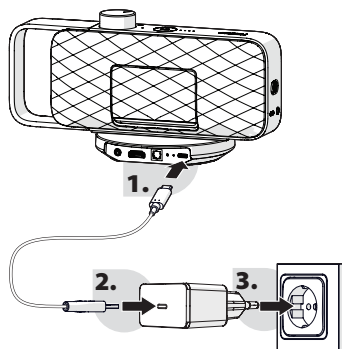


2a

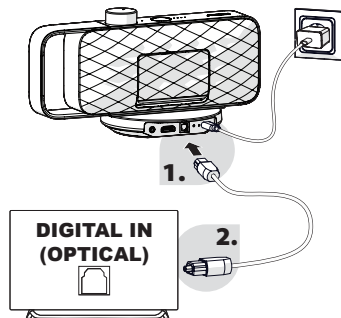
2b

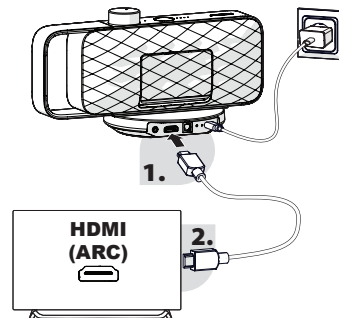
2c

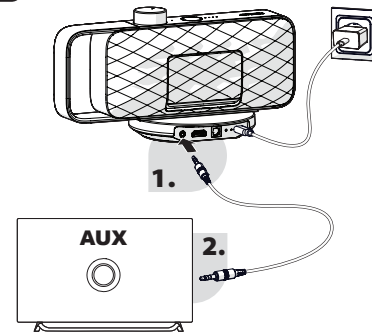
3

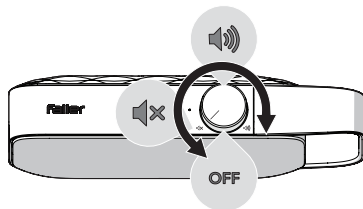4

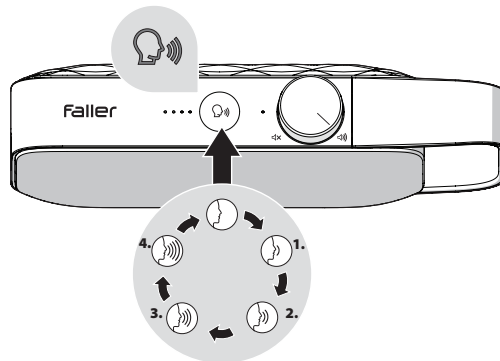5

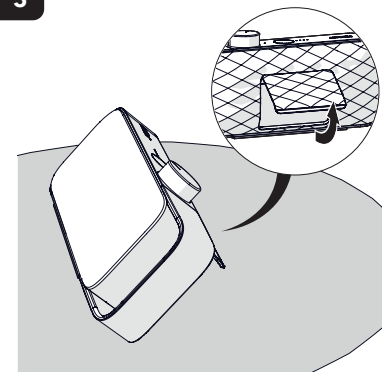**SICHERHEIT / SAFETY / SÉCURITÉ / VEILIGHEID /
SICUREZZA / SEGURIDAD / BEZPIECZEŃSTWO /
SÄKERHET / SIKKERHED / 안전**

DE

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OSKAR is your personal TV voice enhancer, a wireless speaker that brings the sound of your television directly to where you sit, with voices that are clearer, more natural, and easier to follow.

OSKAR analyses your TV's audio in real time thanks to the sonoro Clearvoice Technology®, enhancing speech frequencies while reducing disruptive background noise. The result is a comfortable listening experience at normal volume levels, without affecting the TV sound for others in the room.

OSKAR is designed to replace your TV's integrated speakers, which should be muted during use. It is an assistive listening device that optimises the audio signal from your television. It does not treat or correct any degree of hearing loss, and is intended for private indoor use only.

Please read through this manual before first use so that you can enjoy the OSKAR to the full. If you need help at any point, our customer service team is always happy to assist.

ABOUT THIS MANUAL

This document contains all information for the correct, safe, and efficient use of the OSKAR. Keep this manual for future reference, and pass it on if you transfer the device to another person. Persons who cannot safely operate the device without assistance, including children, should only use it under supervision.

SAFETY

Before using the OSKAR for the first time, please read the safety instructions carefully. They can be found at the back of this booklet. We accept no liability for damage caused by failure to observe those instructions.

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General	
Operating temperature	5 - 40 °C
Frequency range	2402 - 2480 MHz
Maximum transmission power	< 10 dBm
Maximum operating distance	Approximately 20 - 30m (open area, and subject to the environment)
Auxiliary input	3.5 mm stereo socket
Auxiliary input sensitivity	250 mV
Optical input socket	S/PDIF, linear PCM
Optical input sample rate	32KHz / 44.1KHz / 48KHz / 96KHz / 192KHz
Optical channel	Stereo 16 bit, 24bit
HDMI input socket	HDMI-ARC, linear PCM
HDMI input sample rate	32KHz / 44.1KHz / 48KHz / 96KHz / 192KHz
HDMI channel	Stereo 16 bit, 24bit
Input voltage	5.0 VDC


Voice enhancer	
SV-200	
Dimensions (W × D × H)	240 × 60 × 104 mm
Weight	660 grams
Battery type	Lithium-ion rechargeable, C0406C6, 11NR19/66
Battery capacity	3.6V, 3,300 mAh, 11.88Wh
Charging time	Approximately 4 hours
Playing time	Up to 20 hours
Rated power	2 × 5W
Power consumption in Off mode	< 0.3 W
Time to off without signal	15 minutes

Base station	
SV-201	
Dimensions (W × D × H)	121 × 121 × 26.5 mm
Weight	200 grams
Power consumption in another condition (low power mode)	< 0.3 W

AC/DC adapter	
Manufacturer	Ten Pao Industrial Co., Ltd. Room 15, 6/F., Kwong Sang Hong Centre, 151-153 Hoi Bun Road, Kwun Tong, Kowloon, Hong Kong
Model number	S015BT0500300
Input voltage	100 - 240 VAC; 50Hz / 60Hz
Output voltage	5.0 VDC
Output current	3.0 A
Output power	15.0 W
Average active efficiency	> 85.1% @115VAC, 60Hz / @230VAC / 50Hz
Efficiency at low load (10%)	~ 69 %
No-load power consumption	< 0.10 W

The marking information is located on the bottom of the apparatus.

Specifications are subject to change without notice.

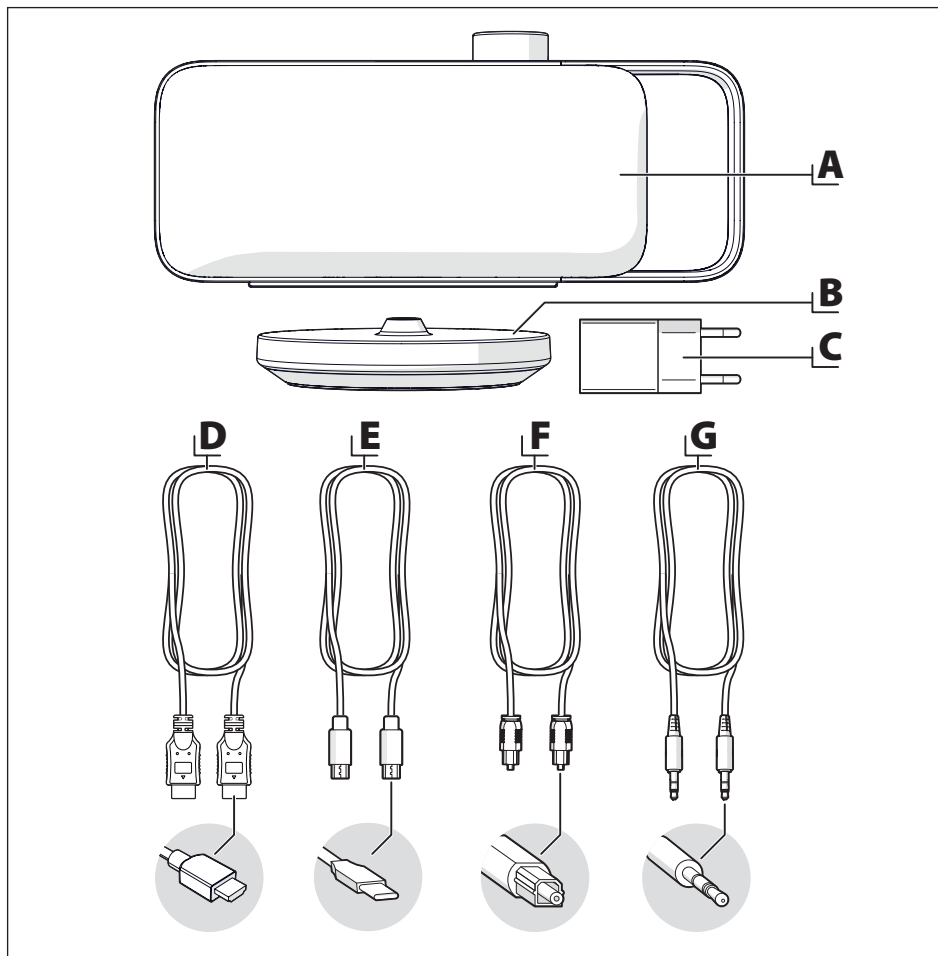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SV-2000-100	SV-2000-200
The power delivered by the charger must be between 5 and 15 watts required by the radio equipment and to achieve the maximum charging speed.	

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INSTALLATION

1. Remove the packaging and check the contents.
2. Place the base station **B** on a flat, stable surface close to your TV.

CHARGING THE OSKAR

Fully charge the OSKAR before first use.

The charging process takes approximately 4 hours. A fully charged battery provides up to 20 hours of operation, depending on the usage conditions.

1. Connect the USB-C charging cable **E** to the USB-C port **14** on the base station **B**.
2. Connect the other end of the USB-C charging cable **E** to the USB-C power adapter **C**.
3. Plug the USB-C power adapter **C** into a wall socket.
4. Place the OSKAR on the base station to start charging. The OSKAR does not need to be switched on during charging. If the speaker is switched off while charging, the status indicator **3** pulses white to show that the battery is being charged;

once the battery is fully charged, the status indicator **3** switches off. If the status indicator **3** lights red, the battery level is below 20% and the OSKAR must be recharged.

Note: Even when the indicator **3** is turning red, you can still use OSKAR for about 2 hours, depending on the volume. Please charge OSKAR immediately afterwards.

CONNECTING THE OSKAR

The base station of the OSKAR provides several connection options for connecting to a TV.

We strongly recommend using the HDMI (ARC) connection. Alternatively, you can connect your TV to the base station using optical connection or AUX connection.

If multiple cables are connected, the base station automatically selects the input in the following order: HDMI, Optical, AUX.

Note: Always connect your TV to the base station **B**, never to the OSKAR itself.

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HDMI (ARC)

1. Connect one end of the HDMI cable **D** into the HDMI port **10** on the rear of the base station **B**.
2. Connect the other end of the HDMI cable **D** into the HDMI (ARC) port of your TV.
3. Make sure that you use the dedicated HDMI port on your TV labelled with **HDMI (ARC)** or **HDMI (eARC/ARC)**.
4. Open your TV's audio settings and set the sound output to HDMI (ARC).
5. Set the TV's digital audio output to PCM in the TV sound settings. If the output is not set to PCM, the OSKAR may not produce sound.

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With this connection type, the TV's integrated speakers should be muted automatically. If this is not the case, there may be a delay between the audio from the OSKAR and the TV's audio. Please mute the TV audio or manually set the TV volume to 0.

When you turn off the OSKAR, your TV may switch back to its integrated speakers.

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OPTICAL CONNECTION

1. Remove the protective caps from both ends of the optical audio cable **F**.

Note: Handle the cable carefully and do not bend it sharply, as this may damage the cable or affect audio transmission.

2. Connect one end of the optical audio cable **F** to the DIGITAL IN port **11** on the rear of the base station **B**.
3. Connect the other end of the optical audio cable **F** to the optical audio output on your TV.

Note: Insert the connector carefully until it is fully seated. Do not use force when connecting the cable.

4. Open your TV's audio settings and set the sound output to the OPTICAL connection.
5. Set the digital audio output format to PCM. If the digital audio output is not set to PCM, no sound may be reproduced through the OSKAR.

Note: Depending on the TV manufacturer and model, the term "PCM Stereo" or a similar phrase may be used. For more information on adjusting your TV settings, please refer to your TV's user manual.

The output from the TV itself may be not muted while using this connection and you may experience a latency between the sound from OSKAR and the sound from the TV. Please either mute the TV sound or manually set the volume of the TV to 0.

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EN AUX

1. Connect one end of the 3.5 mm audio cable **G** to the AUX IN port **9** on the rear of the base station **B**.
2. Connect the other end of the 3.5 mm audio cable **G** to the headphone or AUX audio output port on your TV.
3. Open your TV's audio settings and set the sound output to the AUX connection or to the connected headphones/audio output, depending on the TV model. Set the volume of the TV to a medium level so that the sound can be transmitted to the OSKAR at a sufficient volume.
4. Set the TV volume to a medium level to ensure that the audio signal is transmitted to the OSKAR at a sufficient volume level.

Note: For more information on adjusting your TV settings, refer to your TV's user manual.

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PREPARATION

Once the voice enhancer **A** has been fully charged (the status LED ③ stops blinking white and remains off) for the first time, the voice enhancer **A** can be removed from the base station **B** and placed at the preferred listening position for use.

Note: The voice enhancer **A** can also be charged (separately from the base station) via the integrated USB-C port ⑥. You can generally also use OSKAR while it is charging.

SWITCHING ON THE OSKAR

1. Turn the On/Off/Volume dial ④ clockwise until you hear a click. The status LED ③ will light up white.
2. Adjust the On/Off/Volume dial ④ to a comfortable listening volume.

Note: To reduce power consumption, the base station and the OSKAR automatically switch to low power mode and off mode after 15 minutes if the TV is switched off or if no audio signal is received. To restart the playback, switch off the OSKAR and switch it on again.

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EN

LISTENING MODE

The Listening mode function controls the width of the sound field.

Set the switch **5** to Single listener to narrow the sound field. The speaker focuses the sound in front of the unit. Use this setting when one person is listening. This improves speech clarity at the main listening position.

Set the switch **5** to Multiple listeners to widen the sound field. The speaker spreads the sound over a larger area. Use this setting when more than one person is listening. This provides a more even sound level across the room.

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EN

SPEECH ENHANCEMENT

The OSKAR uses unique technology and algorithm to improve your understanding of dialogue. You hear voices more clearly, while disruptive ambient noises are minimised.

1. Press the speech enhancement button ② to activate this mode. The speech status indicator ① lights up to show that speech enhancement is active.
2. Press the speech enhancement button ② repeatedly to cycle through the different speech enhancement levels. The speech enhancement indicator LED ① indicates which mode is active.

Note: The last used enhancement level is memorised when the unit is switched off and will be restored when the unit is switched on again.

	Speech enhancement off (acoustically optimised TV sound)
	Low speech enhancement (optimised for music and news programmes)
	Medium speech enhancement (recommended for mild hearing loss)
	Strong speech enhancement (recommended for moderate hearing loss)
	Maximum speech enhancement (recommended for pronounced hearing loss)

Note: OSKAR was developed primarily for people with hearing loss. Please note, therefore, that the overall audio quality changes as you increase the intensity of the speech enhancement.

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EN

HEADPHONES

For private listening, headphones can be connected to the OSKAR as needed.

The voice enhancement is then also active through the headphones; OSKAR's speaker is muted when headphones are plugged in.

Note: We recommend using headphones with a 32 ohms impedance to ensure an adequate listening level. Using headphones with a higher impedance will result in a lower listening level.

Before using the headphones, set the volume to the lowest level to avoid exposure to an excessively high sound level.

1. If the OSKAR is already switched on, turn the On/Off/Volume dial ④ counterclockwise to the lowest setting.
2. Connect the 3.5 mm jack plug of the headphones to the headphone port ⑦ on the OSKAR.
3. Adjust the On/Off/Volume dial ④ to a comfortable listening volume.

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EN STATUS INDICATORS

BASE STATION

The pairing indicator LED 13 indicates the current operating status and function of the base station **B**.

Pairing LED state	Meaning
Flashing (slow)	No connection to the voice enhancer
Flashing (fast)	Pairing in progress
Solid	Connected, normal operation

VOICE ENHANCER

The status LED indicates OSKAR's current operating status.

Status LED state	Meaning
Flashing (slow)	No connection to the base station.
Flashing (fast)	Pairing in progress.
Solid	Connected, during operation.
Solid red (while operation)	Battery lower than 20 %. Please charge the OSKAR.
Flashing slow (if switched off on the base station)	Active charging; the indicator turns off when fully charged.

Note: In addition, the speech enhancement LEDs 1 indicate the selected speech enhancement level. This is described in Chapter "Speech enhancement".

RESET AND PAIRING

RESTART BASE STATION

Restarting the base station **B** can help if the OSKAR isn't playing any sound or if the connection between the base station and the OSKAR is not established.

1. Turn the On/Off/Volume dial 4 counterclockwise until it clicks to switch off the OSKAR.
2. Unplug the USB-C power cable **E** from the USB-C power port 14.
3. Wait approximately 30 seconds.
4. Reconnect the USB-C power cable **E**. The base station **B** will restart.

Note: After restarting, the base station **B** will reconnect to the OSKAR automatically.

PAIRING THE BASE STATION AND THE VOICE ENHANCER

The base station **B** and the OSKAR are paired by default and do not need to be paired initially.

Note: If the base station **B** and the OSKAR do not pair or connect automatically (indicated by the flashing status LED 3), this is usually due to other issues such as interference. It may be caused by other devices in the immediate vicinity that are also using Bluetooth or Wi-Fi. Move the base station **B** further away from these devices. See also Chapter 'Troubleshooting'.

Optionally, you can re-pair the base station and the OSKAR.

1. Place the OSKAR near the base station **B** (same room, a few meters away).
2. Press the pairing button 12 on the base station **B**. The status LED 13 will begin to flash rapidly (pairing mode).
3. Turn the On/Off/Volume dial 4 clockwise until it clicks to switch on the OSKAR.

4. Press and hold the speech enhancement button ② on the OSKAR for about 5 seconds until the status LED ③ starts flashing faster.
5. Wait until both LEDs ③ ⑬ are lit steadily. The base station and the OSKAR are now paired.

Note: If pairing fails, repeat the steps. Make sure the OSKAR is sufficiently charged and located near the base station **B**. A failed pairing attempt will prevent the devices from connecting. Repeat the pairing process until it is successful.

MAINTENANCE

CLEANING

Only clean the OSKAR on the outside with a soft, lint-free cloth or a slightly dampened rag. Do not use any wax sprays, harsh cleaning agents, cleaning agents that contain solvent or other substances, as they could damage the surface.

BATTERY CARE

The OSKAR uses a modern lithium-ion battery. Compared to older nickel-metal hydride (NiMH) batteries, it is easier to use in everyday life, and the so-called memory effect is practically non-existent with lithium-ion batteries.

The battery's lifespan depends on the number and depth of charge cycles. For a long battery life, it is best to charge the OSKAR when the status LED ③ is red (battery level below 20%). Avoid letting the battery run completely empty and not charging it. If you do not use the OSKAR for an extended period, ideally store it with a moderate charge level and recharge when needed.

BATTERY DISPOSAL OSKAR

This device contains a lithium-ion battery that must be removed from the device before disposal to ensure proper recycling of materials.

Only remove the rechargeable battery for disposal of the device. Ensure that the battery is completely discharged before removal. Failure to do so may result in injury.

Wear protective goggles and protect your hands, fingers, and working surface. Be aware of the risk of injury from breaking or flying parts.


Procedure:

1. Remove the device from the base station, switch it on, and allow it to run until it enters off mode. Repeat this process until the device can no longer be switched on.

The battery is now fully discharged.

2. Insert an opening pick into the gap at the bottom edge of the back cover and gently pry it upward.
3. Once the bottom is loose, you can lift the cover completely off the device.
4. Carefully pull the battery out of the housing using the connection cable.
5. Disconnect the battery connection.
6. Cover the cable connector with adhesive tape to prevent short circuits caused by residual charge.

The rechargeable battery and the remaining device components can now be disposed of in accordance with local regulations.

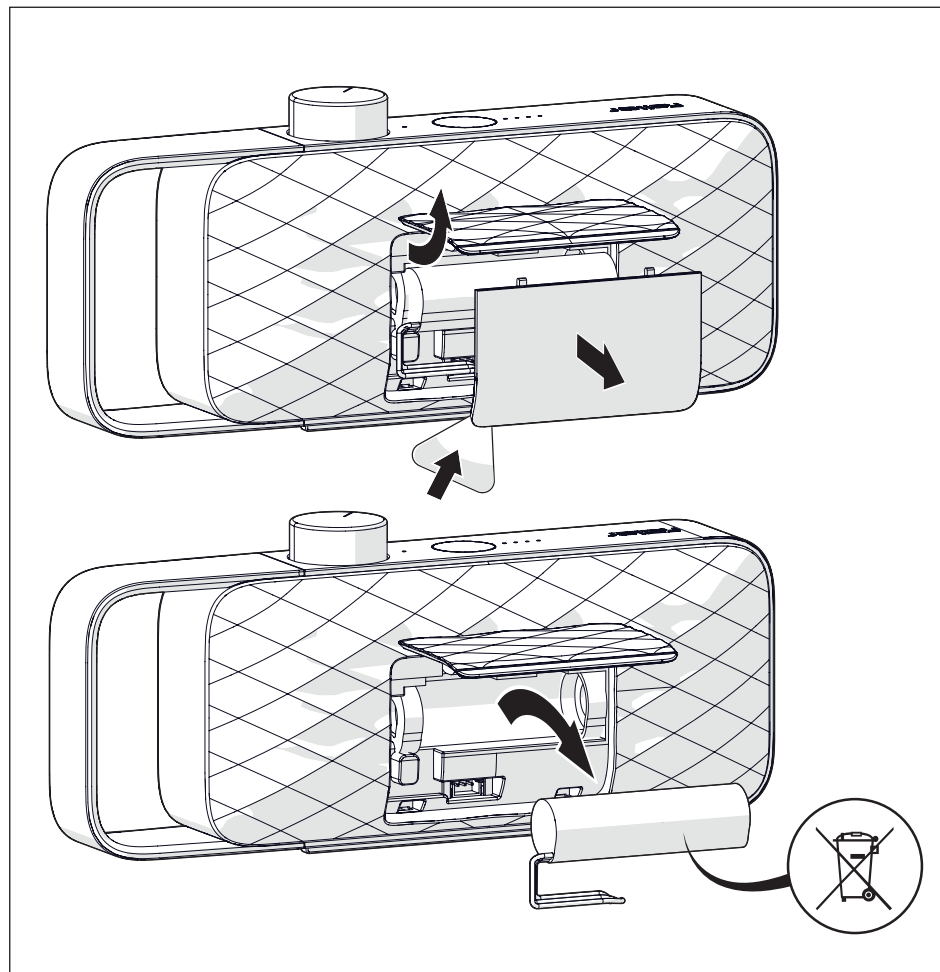TROUBLESHOOTING

Problem	Possible cause	Possible solution
No audio from the OSKAR. • Status LED 3 on the OSKAR: Solid • Pairing indicator LED 15 on base station: Solid	Digital audio format not set to PCM (common after initial installation or switching to a different TV).	Check the audio output format setting in your TV's settings menu. Make sure you have selected PCM as the audio output format on your TV.
	The TV audio has been muted.	Increase the volume on your TV or switch off the mute function.
	No input cable is connected to the base.	Make sure an input cable is connected to the base. Use one of the supported connections: HDMI (ARC), Optical, or AUX.
	The HDMI cable is not connected to the correct HDMI ARC/eARC socket on the TV.	Make sure the HDMI cable is securely connected to the HDMI ARC or HDMI eARC socket on the TV
No audio from the OSKAR. • Pairing indicator LED 15 on base station: Off	Not connected to power socket correctly.	Make sure the USB-C cable E is correctly connected to the base station B and the power supply C .
	Base station B is off or in an unknown state.	Remove the USB cable E (with the power adapter still connected) from the base station and reconnect it to the base station after 30 seconds. The pairing indicator LED 15 should then flash and once connected to the OSKAR, remain lit.
	Power adapter issue.	For further testing, use a different USB-C power adapter with an output power delivery between 5 and 15 watts and a different USB-C cable.
No audio from the OSKAR. • Status LED 3 on the OSKAR voice enhancer is off	If no sound is detected for 15 minutes, the OSKAR and the base station automatically switch to off mode.	To restart the active connection, switch the OSKAR off and then back on.
	The OSKAR is switched off.	Switch the OSKAR on and make sure the status LED becomes solid.
	The OSKAR battery is not sufficiently charged.	Charge the OSKAR voice enhancer by putting it on the base station B or connecting a USB-C charger to the USB-C port 6 directly.

Problem	Possible cause	Possible solution
No audio from the OSKAR. • The pairing indicator LED 15 on the base station B AND the status LED 3 on the OSKAR flash after powering on or intermittently during operation.	The wireless connection between the base station and the OSKAR is unstable or is being interfered with by other wireless sources. Devices such as smart phones, Wi-Fi access points and microwave ovens generate radio waves and can cause radio interference.	Position the OSKAR and the base station at a sufficient distance from such devices. Disable Bluetooth or Wi-Fi on these devices when not needed.
	The OSKAR is too far from the base station, e.g. in another room.	Reduce the distance from the base station B .
Sound interruptions The sound is too quiet, even though the volume on the OSKAR is set to maximum.	The output volume on the TV for the connection being used is set too low.	Check the output level in your TV's sound settings and set it higher or to maximum.
	The output level may be fixed in the TV and is linked to the TV's volume control.	Set the TV volume to a high level (usually $\frac{3}{4}$ of the maximum) and adjust the desired volume on the OSKAR using only its control dial. This also ensures that the OSKAR receives a sufficient signal level to function.
The sound output on the OSKAR is distorted when using the headphones output on your television.	A high output signal is sent to the base station B .	Check the output level in your TV's sound settings and set it to a lower value. Check that the sound does not become distorted even when the volume control on the OSKAR is turned up.
	A high output signal is sent to the base station and the output level is linked to the TV's volume control.	In this case, set the TV volume to a lower level and adjust the desired playback volume on the OSKAR using only its volume control. Check that the sound does not distort even if the OSKAR volume control is turned up.

Problem	Possible cause	Possible solution
A humming noise can be heard during playback.	The AUX cable is not plugged into the AUX or headphone socket on your TV.	Check that you are using the correct socket on your TV.
	The AUX cable is not fully inserted into the base station or the TV.	Make sure that both ends of the cable are fully inserted into the corresponding sockets.
	The socket on the TV is oxidised/ dirty and has poor contact.	Plug the cable into the TV socket several times. Repeatedly plugging it in can improve the contact.
The TV audio and the OSKAR audio are out of sync (when using a optical connection).	The TV audio is not muted.	Mute the TV audio or set the volume to 0. Note: We strongly recommend disabling the TV audio whilst using the OSKAR to ensure optimal speech intelligibility from the OSKAR. If you nevertheless choose parallel audio output, there may be a time delay in the audio output: In your TV's audio settings, enable 'Audio Delay' and find a value so that the sound from the TV is output in sync with the OSKAR.
The battery is not charging	The OSKAR has no charging contact with the base station.	Clean the charging contacts with a soft cloth. To avoid malfunctions, do not press too hard on the charging contacts.

WARRANTY

We guarantee that this device will function properly for 2 years (1 year in North America and Australia) from the date of purchase, provided that the operating instructions in this manual have been observed. This warranty becomes null and void in the event of malfunctions resulting from accidental and intentional damage of any kind, normal wear, excessive wear, negligence, or unauthorised modifications. In addition, this warranty does not apply to deficiencies that only negligibly affect the value or usability of the device. The warranty does not cover disposable or rechargeable batteries. For claims, please contact your specialist dealer or contact our Customer Service department directly. The warranty covers the free repair of all faults resulting from the material or manufacturing process and includes free return shipping. Repair or exchange does not initiate a new warranty period. The warranty period of 2 years (1 year in North America and Australia) from the date of purchase remains decisive.

Please send in the device in the original packaging along with the receipt of purchase. Please do not throw these items away. Claims do not affect and/or cancel the free of charge warranty rights of the customer defined by law.

WARRANTY IF PURCHASED ABROAD

The dealer in the respective country in which the device is purchased provides a warranty for devices that are purchased outside of Germany. You can obtain additional information from your local Faller audio dealer.

DISPOSAL

(To be used in the countries of the European Union and other countries with a separate collection system for these products) This symbol indicates that this product must not be disposed of with other household wastes. Dispose of it in accordance with applicable legal regulations for electric waste. With your contribution to the proper disposal of this product, you protect the environment and the health of other people. Improper disposal endangers the environment and people's health. Material recycling helps to reduce the use of raw materials. To return your used product, you can use the normal return and collection systems or you can contact the shop where you bought the product.

DECLARATION OF CONFORMITY

This product has been manufactured by or for sonoro audio GmbH.

EU importer: sonoro audio GmbH,
Hammer Landstraße 45, 41460 Neuss,
Germany

sonoro audio hereby declares that the product (SV-200 and SV-201) complies with the Radio Equipment Directive (RED) 2014/53/EU, with Directive 2009/125/EC and with the Restriction of Hazardous Substances Directive (RoHS) 2011/65/EU and 2015/863/EU. If a modification is made to the device that has not been authorised by sonoro, this declaration automatically loses its validity.

Please send questions for the importer or questions regarding product conformity based on European legislation by e-mail to: support@sonoro.com or to the address listed above. The full text of the EU declaration of conformity is available at the following Internet address: manuals.faller-audio.com.

COPYRIGHT

©2026 sonoro audio. All rights reserved. “sonoro”, “faller”, the “sonoro” logo, the “faller” logo and other “sonoro audio” marks and symbols are owned by sonoro audio and are potentially registered. All other trademarks are the property of their respective owner. sonoro accepts no liability for errors in this operating manual. The information contained in the operating manual is subject to modification and errors.

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SYMBOLS USED

⚠ WARNING

WARNING: Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.

⚠ CAUTION

CAUTION: Indicates a potentially hazardous situation, which, if not avoided, may result in minor or moderate injury.

NOTICE

NOTICE: Indicates a situation which, if not avoided, may result in property damage to the product or other property.

	The symbol indicates DC voltage.
	The symbol indicates AC voltage.
	For indoor use only.
	The symbol indicates class II equipment.
	The symbol indicates energy efficiency mark.
	Caution. Please refer for further information to the user manual.

Risk of hearing damage from high volume levels. Exposure to loud sound over a prolonged time can cause permanent hearing loss. Always keep the volume at a level that allows awareness of the surroundings. Always reduce the volume before using headphones. Always avoid high volume during extended use.

⚠ WARNING

Risk of choking due to small parts. Small components and packaging materials can be swallowed and cause serious injury or death. Always keep the device, accessories, and packaging out of reach of children and pets.

Risk of injury due to electromagnetic interference. Magnets in the device can interfere with medical devices such as pacemakers and programmable valves. This can result in serious injury. Always keep the device away from persons with such medical devices. Always consult a doctor before use if a medical device is present.

Risk of explosion or fire due to misuse of the lithium-ion battery. Incorrect handling and replacing of a battery with an incorrect type can cause thermal runaway which will result in fire or explosion leading to death or serious injury. Never short-circuit, dismantle, deform, or remove the battery. Always use the battery only as integrated in the device. Replace the battery of the same or an equivalent type.

Risk of fire or explosion due to improper handling of lithium-ion battery. Disposal of the battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion.

Leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas. The battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas. If the battery compartment on your device does not close securely, stop using it and keep it away from infants and children.

Risk of electric shock due to contact with liquids.

Liquids entering the device can cause electric shock, fire, or serious injury. Always keep the device away from rain, moisture, and dripping or splashing liquids. Never place objects filled with liquid near the device. Always ensure that no liquid enters the device. Never touch the device or USB-C power adapter with wet hands.

Risk of electric shock from exposed live parts.

Opening the device or removing covers exposes internal components and can result in serious injury. Only open the device for disposal as described in the chapter 'Disposal'. Never attempt to repair or service the device yourself. Always contact authorized service personnel.

Risk of fire or electric shock due to damaged components.

Damaged cables, adapters, or internal parts can cause fire or serious injury. Always stop using the device if it is damaged, dropped, exposed to moisture, or does not operate correctly. Always have repairs performed by authorized specialists.

Risk of fire or electric shock due to incorrect power supply.

Incorrect voltage or incompatible power supplies can cause overheating, fire, or serious injury. Use only the supplied USB power adapter listed in the operating manual. Always ensure that the voltage on the type plate matches the wall socket.

Risk of fire due to improper installation of power adapter.

The USB power adapter is used as the disconnect device, the socket-outlet shall be installed near the device and shall be easily

accessible. Loose connections can cause overheating and fire. Always fully insert the USB power adapter into a suitable wall socket. Never use sockets where the plug does not fit securely. Never use extension cables or multi-socket adapters. Do not disconnect the device from the power mains by pulling the cable. Always grab hold of the USB-C power adapter.

Risk of fire due to heat or open flames. Exposure to high temperatures or open flames can cause fire or serious injury. Always keep the device away from heat sources. Never place open flames such as candles near the device.

⚠ CAUTION

Risk of minor injury due to falling device. An unstable placement can cause the device to fall and result in injury or damage. Always place the device on a stable and level surface. Always protect the device from mechanical shocks.

Risk of minor injury due to RF exposure. Close proximity to the antenna can exceed recommended exposure limits. Always maintain a minimum distance of 20 cm between the device and the body during operation.

NOTICE

Risk of product damage due to unsuitable environmental conditions.

Exposure to extreme temperatures can impair function or damage the device. Always operate the device within a temperature range of 5 °C to 40 °C. Never expose the device to extreme heat or cold.

Risk of product damage due to improper handling.

Rough handling or dusty environments can reduce product life. Always handle the device with care. Always use the device in clean and dust-free areas.

Risk of product damage due to mechanical stress.

Pressure on sensitive surfaces can cause

deformation or failure. Always protect the front, edges, and corners from excessive force. Never place heavy objects on the device. Never place the device upside down.

Risk of product damage due to unsuitable operating locations.

Exposure to outdoor conditions or unstable environments can damage the device. Never use the device outdoors, near water, or in high humidity. Never use the device in vehicles, on boats, or in medical facilities. Always keep the device away from weather influences.

Risk of product damage due to unauthorized modifications.

Modifications can impair safety and performance. Never modify the device or accessories. Always use only approved accessories designed for this product.

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